- Ⓛ D Gebrauchsanweisung
- Ⓛ GB Instructions for Use
- Ⓛ F Mode d'emploi
- Ⓛ NL Gebruiksaanwijzing

Ligero

 **DIETZ**[®]
REHA-PRODUKTE

Reutäckerstrasse 12
D-76307 Karlsbad-Iltersbach
Tel. +49 (0) 7248/ 9186 - 0
Fax. +49 (0) 7248/ 9186 - 86

CE

Allgemeines

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Ⓞ Instructions for Use

General

Conformity to EU directive 93/42 for medical products is hereby declared for the rollator described here.

These instructions for use are included in the delivery.

They should be retained by the user and also kept with the product if it is transferred.

We reserve the right to make changes to the design shown in these instructions as a result of technical advances.

Reprinting, translation and copying in any form, including partially, require the written permission of the manufacturer.

Copyright remains with the manufacturer.

These instructions are not subject to any document revision process. The current version can be obtained from us.

Indication and intended purpose

The Ligero rollator can be used as an aid for people with limited mobility up to a maximum body weight of 120 kg and can also be used to a limited extent as a seat for temporary rest purposes.

The basket mounted under the seat is provided for storage and transport of objects and goods.

The Ligero rollator may only be used on solid, even surfaces.

Use for other purposes, in particular use as an aid for climbing steps or for transporting heavy loads is not in accordance with its intended purpose and is dangerous!

Setting up/unfolding the rollator

- Position the rollator in front of you in the intended direction of travel
- Operate the locking brake
- Hold the rollator with one hand on the handle
- With the other hand, press the folding mechanism downwards until you can feel it lock in position
- Fold the seat down
- Fold the backrest into the position for use and then hang the basket onto the frame from the front

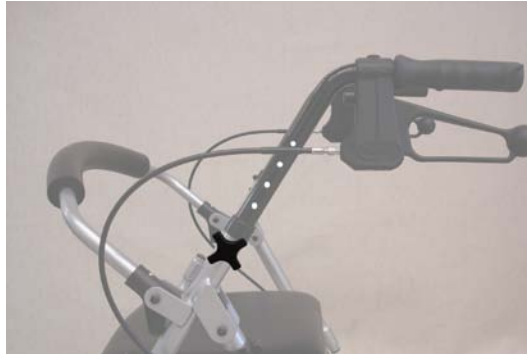
Folding the rollator

- Remove the basket from the frame
- Position the rollator pointing away from you and operate the parking brake
- Fold up the seat
- Hold the rollator with one hand on the handle
- With the other hand, release the folding mechanism by pulling sharply out of its position
- Now pull the rollator completely towards you

Adjusting the push handle

- First operate the locking brake
- Unscrew the bolt and pull the fixing bolt completely out
- Position the push handle at the correct height (recommended: the push handles should be at wrist height)
- Finally, reinsert the fixing bolts and tighten with the star grip nut

Make sure that the bolt goes completely into the hole.

For storing the rollator, you can completely remove the push handles from the frame.

- The brake is used during movement (when pushing) by pulling the brake lever upwards
- Pressing the lever downwards enables the brake to be used as a locking brake (for parking)

Adjusting the brake

- The braking effect can be adjusted directly at the brake lever or at the Bowden cable
- The brake is correctly adjusted when the brake lever and Bowden cable do not exhibit any play and the wheels rotate without any rubbing noises

Safety notes

- Do not hang bags on the handles.
- If using on sloping surfaces, use the main brake.
- Only use the rollator on level, horizontal and solid surfaces.
- Leaning heavily on the handles can cause the rollator to tip backwards.
- When folding the Ligero rollator, clothing and parts of the body can become trapped.
- Please note that padded parts can become hot in direct sunlight and lead to skin injury on contact. Therefore, cover these parts or protect the rollator from direct sunlight.

Cleaning

Cleaning and disinfection should be carried out at regular intervals (a requirement with a change of user). Use a mild soap solution without scouring additives for cleaning. Disinfection can be carried out using normal commercial products. Do not use steam cleaning devices, scouring or corrosive products!

Maintenance

The Ligero rollator is maintenance-free. If you notice a reduction in braking, difficulty moving the wheels or other functional limitations, the rollator should be taken to a specialist for repair.

Further use

When the rollator is transferred to another person, ensure that all technical documentation necessary for its safe use is given to the new user. The rollator must be checked, cleaned and put into a perfect condition by a specialist for continued use.

The Ligero rollator is maintenance-free. If you notice a reduction in braking, difficulty moving the wheels or other functional limitations, the rollator should be taken to a specialist for repair.

Scope of Delivery

- Ligero rollator
- shopping basket
- instructions for use
- cane holder

Warranty

This product is guaranteed for a period of 24 months and covers all defects in the product that are shown to be caused by material or production defects.

Technical data

| | |
|---------------------------|---|
| Maximum load: | 120 kg / 3 kg shopping basket |
| Weight empty: | 6.8 kg |
| Overall width: | 550 mm |
| Overall depth: | 670 mm |
| Overall height: | 820 – 930 mm |
| Distance between handles: | 480 mm |
| Turning circle: | 710 mm |
| Seat height: | 510/560 mm |
| Adjustment: | Push handle: 820 – 930 mm (5 position) |
| Tyres: | Ø 195 x 37 mm, solid |

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